

## BACOBAN<sup>1</sup> 1%

## **DISINFECTION & CLEANING READY-TO-USE**

## WATER-BASED

Alcohol-Free I For surfaces, medical fittings and equipment

Residual antimicrobial effect for up to 10 days

According with guideline 93/42/EEC; MDD I excellent skin tolerance I high cleaning effect

Effective against Coronavirus I practical applicability tested in the 4-field-test

100 g of solution containing: Benzalkonium chloride 0,78 g, sodium pyrithione 0,075 g, polycondensates, perfume substances, purified water.

Ingredients according to regulation (EC) No 648/2004: less than 5 % cationic surfactants, disinfectants, perfumes.

Does not contain formaldehyde. Does not contain phenols.

Efficacy and application:

Tested in accordance with EN guidelines:

EN 1040: 5 min EN 1275: EN 1276: 5 min 5 min EN 1650: 5 min EN 13697: 15 min EN 13727: 5 min

EN 13624: 5 min (C. albicans) EN 13624: 5 min (T. interdigitale)

EN 14561: 60 min

EN 14562: 60 min EN 16615: 5 min (4-field-test) Virucidal activity against enveloped viruses according to EN 14476 under dirty conditions: 5 min (incl. Coronavirus, HIV, HBV, HCV, Influenza,

Herpes, Ebola virus,...

EN 14476: Rotavirus 5 min Norovirus 30 min

Human papillomavirus 60 min

Residual antimicrobial effect for up to 10 days (tested according to

Dermatologically tested (epicutaneous test with 30 test persons): excellent skin tolerance

Important!

The residual effect of Bacoban® DL is no excuse for not carrying out necessary disinfection measures in cases of visible and/or questionable contamination!

Safety notice

Safety data sheet available on request.

**Storage**Store Bacoban®DL at +5 bis +25°C. Protect from frost and direct sunlight.

Type of delivery Product	Packaging unit	Item number
Bacoban®DL Bottle Bacoban®DL Canister Bacoban®DL Canister		BAC-B2-1 L BAC-B2-5 L BAC-B2-10 L

## Intended use:

Ready-to-use disinfectant product with residual effect. Does not contain Aldehyde or Phenol.

Directions for use: Spread Bacoban® DL evenly over the surface with a cloth and leave to dry. Do not rinse.

Use disinfectants safely. Always read the label and product information before use.

Safety instructions:

Wear gloves when using disinfectant products. Keep out of reach of

Can be used on all surfaces. For possibly sensitive surfaces, we recommend testing for compatibility first on a less visible place. To be kept in a dry, aired place. Close properly after use. Recycle and dispose of the container in accor-dance with current legislation, preferably via an approved collector or company. Do not use after the sell-by date. A safety sheet can be obtained on request.

Biocide Reg.No: N-93957

