

BACOBAN^{DL} 1%

DISINFECTION & CLEANING

READY-TO-USE

WATER-BASED

Alcohol-Free | For surfaces, medical fittings and equipment

Residual antimicrobial effect for up to 10 days

According with guideline 93/42/EEC; MDD | excellent skin tolerance | high cleaning effect

Effective against Coronavirus | practical applicability tested in the 4-field-test

Ingredients:

100 g of solution containing: Benzalkonium chloride 0,78 g, sodium pyrithione 0,075 g, polycondensates, perfume substances, purified water.
 Ingredients according to regulation (EC) No 648/2004: less than 5 % cationic surfactants, disinfectants, perfumes.
 Does not contain formaldehyde. Does not contain phenols.

Efficacy and application:

Tested in accordance with EN guidelines:

EN 1040: 5 min
 EN 1275: 5 min
 EN 1276: 5 min
 EN 1650: 5 min
 EN 13697: 15 min
 EN 13727: 5 min
 EN 13624: 5 min (C. albicans)
 EN 13624: 5 min (T. interdigitale)
 EN 14561: 60 min
 EN 14562: 60 min
 EN 16615: 5 min (4-field-test)
 Virucidal activity against enveloped viruses according to EN 14476 under dirty conditions: 5 min (incl. Coronavirus, HIV, HBV, HCV, Influenza, Herpes, Ebola virus,...)
 EN 14476: Rotavirus 5 min
 Norovirus 30 min
 Human papillomavirus 60 min

Residual antimicrobial effect for up to 10 days (tested according to ASTM E 2180).

Dermatologically tested (epicutaneous test with 30 test persons): excellent skin tolerance

Important!

The residual effect of Bacoban® DL is no excuse for not carrying out necessary disinfection measures in cases of visible and/or questionable contamination!

Safety notice

Safety data sheet available on request.

Storage

Store Bacoban®DL at +5 bis +25°C. Protect from frost and direct sunlight.

Type of delivery

Product	Packaging unit	Item number
Bacoban®DL Sachets	50/100 pcs	BAC-B4-SAC

Intended use:

Ready-to-use disinfectant product with residual effect. Does not contain Aldehyde or Phenol.

Directions for use: Spread Bacoban® DL evenly over the surface with a cloth and leave to dry. Do not rinse.

Use disinfectants safely. Always read the label and product information before use.

Safety instructions:

Wear gloves when using disinfectant products. Keep out of reach of children.
 Can be used on all surfaces. For possibly sensitive surfaces, we recommend testing for compatibility first on a less visible place. To be kept in a dry, aired place. Close properly after use. Recycle and dispose of the container in accordance with current legislation, preferably via an approved collector or company. Do not use after the sell-by date. A safety sheet can be obtained on request.

Biocide Reg.No: N-939978

B4